

Lite Instructions




Robot Status

1.0 Press the icon “i” which is located at the bottom right corner.




1.1 Please verify that the Robot indicates “Ready”.

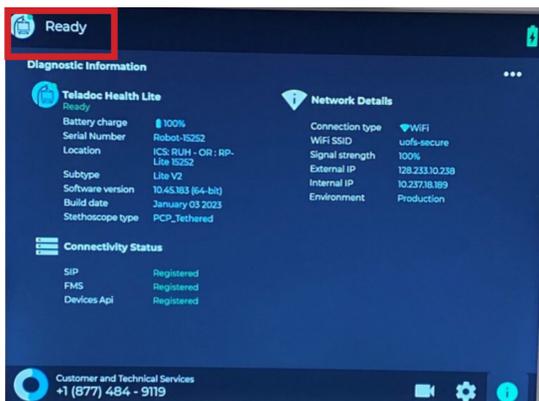


Robot Modes

3.0 Press the gear icon.

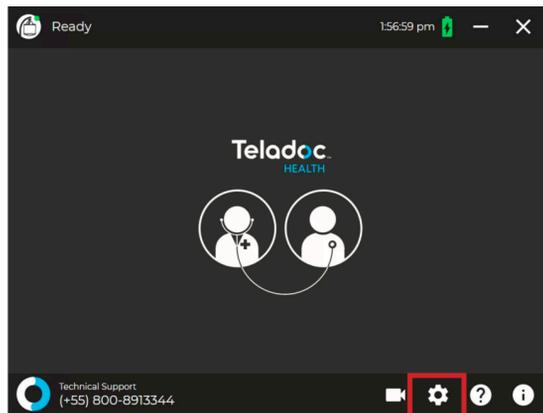


3.1 Cleaning Mode allows you to clean the screen (Robot’s head). Please use a damp cloth, 70% alcohol or regular wipe removing excess water before wiping.

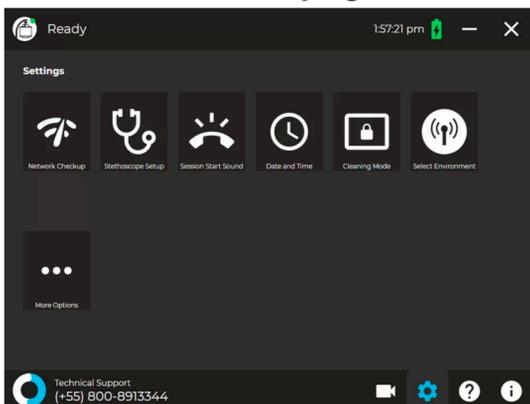

Stethoscope

4.0 Stethoscope is inside the Robot Storage compartment which is located at the back.

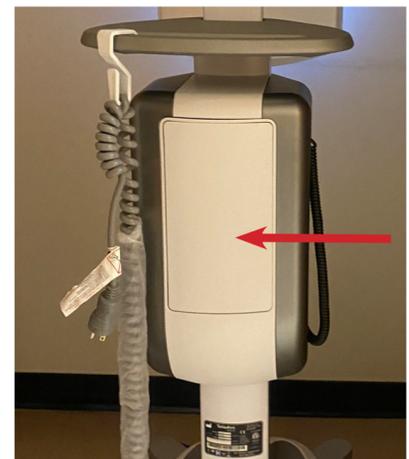


4.1 Press the compartment to open the lid.



4.2 Connect the stethoscope into the USB port which is located on the expansion panel.

Note: Please UNPLUG and SANITIZE the stethoscope after use; store it back in the storage compartment.

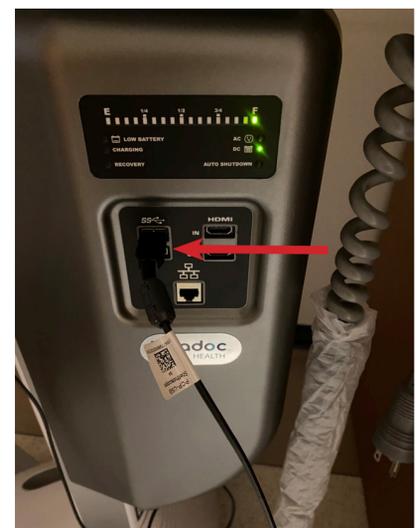

Rebooting Process

2.0 Press the power button which is located at the back of the Robot’s head.



2.1 The Monitor’s screen should be black, if it is not, please press the power button again until monitor is black..

2.2 Press the power button. Robot should start rebooting. Head will move to start the system.

3.2 Stethoscope Setup allows you to connect a stethoscope.

Note: The USB stethoscope must be selected.

